## OMRON PFM20 Peak Flow Meter

### Single Patient Use Instructions for Use

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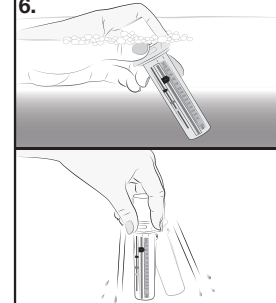
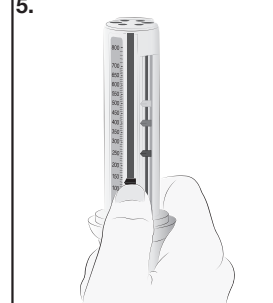
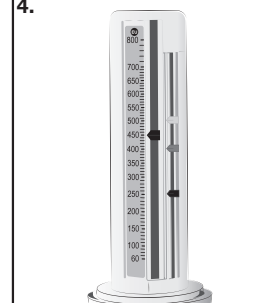
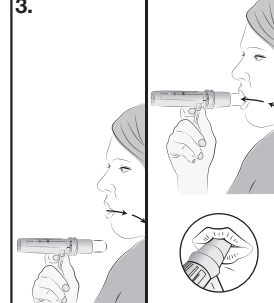
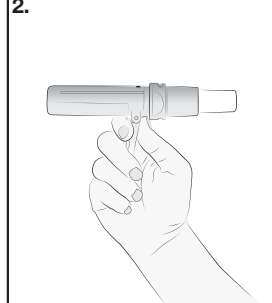
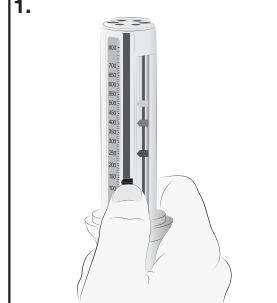
### EN

Your OMRON PFM20 Peak Flow Meter will help you to monitor your asthma by measuring your peak flow. This leaflet explains how to use and care for your peak flow meter and how to record the readings. With careful use the OMRON PFM20 would be expected to last for two years or approx. 2,000 uses.

**Medical Purpose:** The OMRON PFM20 Peak Flow Meter is a device used to determine lung function by measuring the maximum rate at which air is exhaled from the lungs. This is also referred to as the Peak Expiratory Flow (PEF) or more simply, the Peak Flow.

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#### Technical Data

Product category:	Peak flow meters
Product description:	Peak flow meter
Model:	PFM20
Operating	<p>temperature: 10 to 35°C (50 to 95°F)</p> <p>humidity: 30 to 75%RH (not condensed)</p> <p>air pressure: 850 - 1060hPa</p>
Storage and Transport temperature: humidity:	-5 to 50°C (23 to 122°F) <p>10 to 95%RH (not condensed)</p>
Accuracy:	Better than 10% or 10L/min.
Max. resistance to flow:	0.36 kPa/L/s
Weight:	44g
Dimensions:	44 (W) x 139 (H) x 35 (D) mm
Durable period:	2 years
Package content:	1 Adult / Child PFM20 Peak Flow Meter complete with optional zone markers, universal mouthpiece and instructions with Peak Flow Diary.

#### Cleaning Instructions

(for Single Patient Use)

Immerse the peak flow meter in warm (but not hot) mild detergent solution for 2-3 minutes (maximum 5 minutes). Agitate the meter to ensure thorough cleaning. Rinse in clean warm water and shake gently to remove any excess water.

Allow to dry thoroughly before using again.

**Disposal Note:** The OMRON PFM20 peak flow meter can be disposed of as normal household waste.

**6** Return the pointer to the bottom of the scale (nearest the mouthpiece) and repeat the procedure twice more to obtain three readings. Mark the highest of the three readings on your Peak Flow Diary provided overleaf (Refer to 'How to record your peak flow').

#### How to Record Your Peak Flow

It is important to keep a record of your peak flow. Three readings should be taken twice a day, first thing in the morning and at bedtime unless otherwise directed by your healthcare professional.

***With a dot record the highest of the three readings on the Peak Flow Diary, both in the morning and at bedtime, every day.***

By joining the dots (see example, overleaf) you can see how your peak flow varies from day to day.

Make sure to measure and note your peak flow if you suddenly become wheezy or breathless at any time of the day or night.

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#### Using the Zone Monitoring System

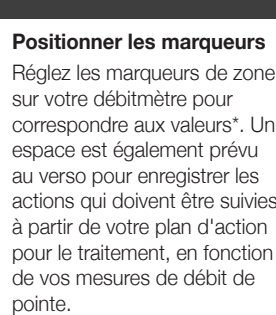
The use of a 3 zone monitoring system set up like traffic lights uses coloured plastic clips (green, yellow, red) as zone markers and has proven useful to patients monitoring their asthma with peak flow readings.

Once your green, yellow and

The general principle of the zone monitoring system is as follows:

<b>Zone</b>	<b>Condition</b>	<b>Reading</b> (% of best reading)	<b>Action</b>
<b>Green</b>	<b>Go</b>	80-100%	<i>Go ahead with normal activities.</i>
<b>Yellow</b>	<b>Caution</b>	50-80%	<i>Use caution in your activities. Refer to your treatment plan for actions to be taken.</i>
<b>Red</b>	<b>STOP</b>	Less than 50%	<i><b>Medical alert! Seek immediate medical attention.</b></i>

These would normally be set by your Doctor or Nurse

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red zones have been determined, record their values in the Treatment Action Plan overleaf.

#### Positioning the Markers

Set the zone markers on your meter to correspond with the values\*. Space has also been provided overleaf to record the actions to be taken, from your Treatment Action Plan, based on your peak flow readings.

The general principle of the zone monitoring system is as follows:

<b>Zone</b>	<b>Condition</b>	<b>Reading</b> (% of best reading)	<b>Action</b>
<b>Green</b>	<b>Go</b>	80-100%	<i>Go ahead with normal activities.</i>
<b>Yellow</b>	<b>Caution</b>	50-80%	<i>Use caution in your activities. Refer to your treatment plan for actions to be taken.</i>
<b>Red</b>	<b>STOP</b>	Less than 50%	<i><b>Medical alert! Seek immediate medical attention.</b></i>

These would normally be set by your Doctor or Nurse

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\* For example, if you and your physician or other licensed healthcare professional determine your individual best reading is 500 LPM, then your Green Zone is 400-500 LPM (80-100%). Set the green marker at 500. Your Yellow Zone is 250-400 LPM (50-80%). Set the yellow marker at 400. Your Red Zone is 0-250 (less than 50%). Set the red marker at 250.

#### ⚠ Caution

Do not attempt to remove the coloured clips once they have been attached to the OMRON PFM20. They are designed to stay in their specified position and can not be moved.

**This device is not a toy.**
Danger of small parts. Choking hazard.

#### Troubleshooting

Should you take your peak flow measurement and get an unusually low reading, it could

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mean that your peak flow meter is broken or that the reading is accurate showing your asthma is getting worse.

Check to make sure that the meter is not broken by sliding the pointer along the slot, making sure that it moves freely. If you feel comfortable that you are following the instructions correctly and that

your meter is not broken, contact your healthcare professional for advice.

No matter what your peak flow readings are, if you have symptoms such as chest tightness, shortness of breath, coughing or wheezing you should contact your healthcare professional for advice.

Should you take your peak flow measurement and get an unusually low reading, it could

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